



**PHILIPS**

Hospital  
respiratory care

# Confidence

in your patient's comfort







Philips Respironics patient interfaces and circuits for noninvasive ventilation

# Delivering **exceptional** NIV

Philips Respironics masks are designed to support enhanced NIV. From comfort for your patients to ease of use for you, our interfaces are specially designed to meet the rigors of acute care environments.

# Table of contents

---

	Total face masks	4
	Oro-nasal masks	8
	Nasal masks	16
	Packs and accessories	19
	Circuits and connections	26
	Trilogy accessories	27

---



## Total face masks

Total face masks allow quick fitting and eliminate nasal bridge challenges by sealing around the perimeter of the face, where patients have less pressure sensitivity and smoother facial contours.

### **Mask selection criteria**

- Rapid application
- Mouth breather
- Facial abnormalities
- Lack of teeth





# Philips Respironics Total



### Key benefits

- Clear, unobstructed view
- Perimeter facial contact increases comfort and simplifies fitting
- Digital Auto-Trak compatibility enhances NIV performance

For headgear, see page 26

### Total

## Philips Respironics Total

Description	Quantity	Product number	Part number
Total face mask	1	302433	989805609231
Total face mask head strap	1	1003052	453561517111
Total face mask head strap	9	1010609	989805610031



# Philips Respironics PerforMax single-use



### Key benefits

- Available for patients 7 years or older and > 40 lb (20 kg)
- Perimeter facial contact increases comfort and simplifies fitting
- Clear, unobstructed view

Compatible with a leak 2 entrainment elbow (EE), standard elbow (SE), and bronchoscopy elbow (BE)  
 For accessories, see page 23  
 For headgear, see page 26

### PerforMax EE leak 2

## Philips Respironics PerforMax single-use

Description	Quantity	Product number	Part number
PerforMax with EE leak 2 elbow, small*	1	1061619	989805627831
PerforMax with EE leak 2 elbow, large	1	1061620	989805627841
PerforMax with EE leak 2 elbow, extra large	1	1080426	989805634831
PerforMax with EE leak 2 elbow, small*	10	1052531	989805611241
PerforMax with EE leak 2 elbow, large	10	1052532	989805611251
PerforMax with EE leak 2 elbow, extra large	5	1080424	989805634811
PerforMax with SE standard elbow, small*	1	1061621	989805627851
PerforMax with SE standard elbow, large	1	1061622	989805627861
PerforMax with SE standard elbow, extra large	1	1080517	989805634841
PerforMax with SE standard elbow, small*	10	1052533	989805611261
PerforMax with SE standard elbow, large	10	1052534	989805611271
PerforMax with SE standard elbow, extra large	5	1080425	989805634821

\* Intended for use by patients 7 years or older and > 40 lb (20 kg)



# Philips Respironics PerforMax multi-use



### Key benefits

- Available for patients 7 years or older and > 40 lb (20 kg)
- Perimeter facial contact supports comfort and simplifies fitting
- Clear, unobstructed view

Compatible with a leak 2 entrainment elbow (EE), standard elbow (SE), and bronchoscopy elbow (BE)  
 For accessories, see page 23  
 For headgear, see page 26

### PerforMax SE

## Philips Respironics PerforMax multi-use

Description	Quantity	Product number	Part number
PerforMax with EE leak 2 and SE standard elbow, small*	1	1049037	989805611201
PerforMax with EE leak 2 and SE standard elbow, large	1	1049038	989805611211
PerforMax with EE leak 2 and SE standard elbow, extra large	1	1080423	989805634801
PerforMax with EE leak 2 elbow, small*	1	1047426	989805617081
PerforMax with EE leak 2 elbow, large	1	1047427	989805617091
PerforMax with EE leak 2 elbow, extra large	1	1080419	989805634761

\* Intended for use by patients 7 years or older and > 40 lb (20 kg)

## Oro-nasal masks

Oro-nasal masks are the standard of care for hospital NIV, and the Philips Respironics family includes a comprehensive selection of systems with features that address patient comfort, mask fitting, and ventilator compatibility.



### Mask selection criteria

- Mouth breather
- Long-term NIV
- Two cushion styles for mask rotation

## Patient comfort made easy

CapStrap headgear provides excellent stability and simplifies initial setup by keeping the mask in place while you adjust the patient's straps. When you need to interrupt treatment for oral care and medications, the CapStrap headgear allows quick removal and simple reapplication. CapStrap headgear is available on several Philips Respironics masks:

- AF541
- AF531
- AF811
- PerformaTrak





# Philips Respironics AF541



AF541 with four-point headgear, EE leak 1

## Key benefits

- The under-the-nose cushion can be used to off load mask pressure at bridge of nose
- Push-button forehead pad adjustment, easy-to-use CapStrap headgear, and snap-in-place elbows and talon clips contribute to efficient workflow
- Discharge kit configures mask to work with home NIV systems
- CleanClip system supports your infection control efforts
- Discharge kit for patient transition to lower acuity setting

For accessories, see page 23  
For headgear, see page 25

## Philips Respironics AF541

Description	Quantity	Product number	Part number
AF541 with EE leak 1 elbow, CapStrap headgear, small*	10	1120925	989805653931
AF541 with EE leak 1 elbow, CapStrap headgear, medium*	10	1120926	989805653941
AF541 with EE leak 1 elbow, CapStrap headgear, large	10	1120927	989805653951
AF541 with EE leak 1 elbow, CapStrap headgear, extra large	10	1120928	989805653961
AF541 with EE leak 2 elbow, CapStrap headgear, small*	10	1120935	989805654011
AF541 with EE leak 2 elbow, CapStrap headgear, medium*	10	1120936	989805654021
AF541 with EE leak 2 elbow, CapStrap headgear, large	10	1120937	989805654031
AF541 with EE leak 2 elbow, CapStrap headgear, extra large	10	1120938	989805654041
AF541 with SE standard elbow, CapStrap headgear, small*	10	1120945	989805654091
AF541 with SE standard elbow, CapStrap headgear, medium*	10	1120946	989805654101
AF541 with SE standard elbow, CapStrap headgear, large	10	1120947	989805654111
AF541 with SE standard elbow, CapStrap headgear, extra large	10	1120948	989805654121
AF541 with EE leak 1 elbow, four-point headgear, small*	10	1120920	989805653891
AF541 with EE leak 1 elbow, four-point headgear, medium*	10	1120921	989805653901
AF541 with EE leak 1 elbow, four-point headgear, large	10	1120922	989805653911
AF541 with EE leak 1 elbow, four-point headgear, extra large	10	1120923	989805653921
AF541 with EE leak 2 elbow, four-point headgear, small*	10	1120930	989805653971
AF541 with EE leak 2 elbow, four-point headgear, medium*	10	1120931	989805653981
AF541 with EE leak 2 elbow, four-point headgear, large	10	1120932	989805653991
AF541 with EE leak 2 elbow, four-point headgear, extra large	10	1120933	989805654001
AF541 with SE standard elbow, four-point headgear, small*	10	1120940	989805654051
AF541 with SE standard elbow, four-point headgear, medium*	10	1120941	989805654061
AF541 with SE standard elbow, four-point headgear, large	10	1120942	989805654071
AF541 with SE standard elbow, four-point headgear, extra large	10	1120943	989805654081
AF541 size A under-the-nose cushion	10	1120980	989805654271
AF541 size B under-the-nose cushion	10	1120981	989805654281
AF541 size C under-the nose cushion	10	1120982	989805654291
Bronchoscopy elbow	5	1120984	989805654301
AF541 discharge kit, CapStrap headgear, small/medium	5	1120986	989805654311
AF541 discharge kit, CapStrap headgear, large/extra large	5	1120988	989805654331
AF541 discharge kit, four-point headgear, small/medium	5	1120987	989805654321
AF541 discharge kit, four-point headgear, large/extra large	5	1120989	989805654341

\* Intended for use by patients 7 years or older and > 40 lb (20 kg)





# Philips Respironics AF531



AF531 with CapStrap



AF531 EE leak 1

## Key benefits

- CapStrap headgear provides an excellent fit and simple reapplication
- Supports your infection control efforts with the CleanClip system
- You can adapt one mask to multiple ventilators with interchangeable elbows

Compatible with a leak 1 or leak 2 entrainment elbow (EE), EE NIVO elbow (EE/NE), standard elbow (SE), SE NIVO elbow (SE/NE), and bronchoscopy elbow (BE)  
 For accessories, see page 23  
 For headgear, see page 26

## Philips Respironics AF531 with CapStrap single-use

Description	Quantity	Product number	Part number
AF531 single-use with EE leak 1 elbow, CapStrap headgear, small*	1	1061727	989805628611
AF531 single-use with EE leak 1 elbow, CapStrap headgear, medium	1	1061728	989805628621
AF531 single-use with EE leak 1 elbow, CapStrap headgear, large	1	1061729	989805628631
AF531 single-use with EE leak 1 elbow, CapStrap headgear, small*	10	1072627	989805628731
AF531 single-use with EE leak 1 elbow, CapStrap headgear, medium	10	1072628	989805628741
AF531 single-use with EE leak 1 elbow, CapStrap headgear, large	10	1072629	989805628751
AF531 single-use with SE standard elbow, CapStrap headgear, small*	1	1061730	989805628641
AF531 single-use with SE standard elbow, CapStrap headgear, medium	1	1061731	989805628651
AF531 single-use with SE standard elbow, CapStrap headgear, large	1	1061732	989805628661
AF531 single-use with SE standard elbow, CapStrap headgear, small*	10	1072630	989805628761
AF531 single-use with SE standard elbow, CapStrap headgear, medium	10	1072631	989805628771
AF531 single-use with SE standard elbow, CapStrap headgear, large	10	1072632	989805628781
AF531 single-use with EE leak 1/NE nebulizer elbow, CapStrap headgear, small*	10	1072618	989805635001
AF531 single-use with EE leak 1/NE nebulizer elbow, CapStrap headgear, medium	10	1072619	989805635011
AF531 single-use with EE leak 1/NE nebulizer elbow, CapStrap headgear, large	10	1072620	989805635021

\* Intended for use by patients 7 years or older and > 40 lb (20 kg)



# Philips Respironics AF531



AF531 with four-point headgear



AF531 Bronchoscopy\*



AF531 SE

## Key benefits

- Supports your infection control efforts with the CleanClip system
- You can adapt one mask to multiple ventilators with interchangeable elbows

For accessories, see page 23

For headgear, see page 26

## Philips Respironics AF531 with four-point headgear single-use

Description	Quantity	Product number	Part number
AF531 single-use with EE leak 1 elbow, four-point headgear, small**	1	1061718	989805628521
AF531 single-use with EE leak 1 elbow, four-point headgear, medium	1	1061719	989805628531
AF531 single-use with EE leak 1 elbow, four-point headgear, large	1	1061720	989805628541
AF531 single-use with EE leak 1 elbow, four-point headgear, small**	10	1072621	989805628671
AF531 single-use with EE leak 1 elbow, four-point headgear, medium	10	1072622	989805628681
AF531 single-use with EE leak 1 elbow, four-point headgear, large	10	1072623	989805628691
AF531 single-use with SE standard elbow, four-point headgear, small**	1	1061721	989805628551
AF531 single-use with SE standard elbow, four-point headgear, medium	1	1061722	989805628561
AF531 single-use with SE standard elbow, four-point headgear, large	1	1061723	989805628571
AF531 single-use with SE standard elbow, four-point headgear, small**	10	1072624	989805628701
AF531 single-use with SE standard elbow, four-point headgear, medium	10	1072625	989805628711
AF531 single-use with SE standard elbow, four-point headgear, large	10	1072626	989805628721
AF531 single-use with EE leak 1 nebulizer elbow, four-point headgear, small**	10	1061633	989805634971
AF531 single-use with EE leak 1 nebulizer elbow, four-point headgear, medium	10	1061634	989805634981
AF531 single-use with EE leak 1 nebulizer elbow, four-point headgear, large	10	1061635	989805634991

\* Bronchoscopy elbow sold separately, see page 23

\*\* Intended for use by patients 7 years or older and > 40 lb (20 kg)



# Philips Respironics AF531 with leak 2 elbow



### Key benefits

- Leak built into the mask
- Can be used without an additional exhalation port
- Available with four point headgear and CapStrap headgear
- Convenient ten-packs

For accessories, see page 23  
For headgear, see page 26

### AF531 EE leak 2

## Philips Respironics AF531 with leak 2 elbow single-use

Description	Quantity	Product number	Part number
AF531 with EE leak 2 elbow, four-point headgear, small*	1	1061724	989805628581
AF531 with EE leak 2 elbow, four-point headgear, medium	1	1061725	989805628591
AF531 with EE leak 2 elbow, four-point headgear, large	1	1061726	989805628601
AF531 with EE leak 2 elbow, four-point headgear, small*	10	1100393	989805645651
AF531 with EE leak 2 elbow, four-point headgear, medium	10	1100394	989805645661
AF531 with EE leak 2 elbow, four-point headgear, large	10	1100395	989805645671
AF531 with EE leak 2 elbow, CapStrap headgear, small*	10	1100390	989805645621
AF531 with EE leak 2 elbow, CapStrap headgear, medium	10	1100391	989805645631
AF531 with EE leak 2 elbow, CapStrap headgear, large	10	1100392	989805645641

\* Intended for use by patients 7 years or older and > 40 lb (20 kg)





# Philips Respironics AF811



### Key benefits

- Cushion uses a gel pad with a silicone flap for increased comfort
- CapStrap headgear system simplifies mask application and routine patient care
- Available in three sizes to fit a wide patient population

Entrainment elbow (EE)  
 Also available with standard elbow (SE)  
 For headgear, see page 26

AF811 EE

## Philips Respironics AF811 single-use

Description	Quantity	Product number	Part number
AF811 oro-nasal single-use mask EE, CapStrap headgear, small	1	1063047	989805626321
AF811 oro-nasal single-use mask EE, CapStrap headgear, medium	1	1063016	989805626311
AF811 oro-nasal single-use mask EE, CapStrap headgear, large	1	1063015	989805626301
AF811 oro-nasal single-use mask EE, CapStrap headgear, small	5	1063053	989805626381
AF811 oro-nasal single-use mask EE, CapStrap headgear, medium	5	1063052	989805626371
AF811 oro-nasal single-use mask EE, CapStrap headgear, large	5	1063051	989805626361
AF811 oro-nasal single-use mask SE, CapStrap headgear, small	1	1063050	989805626351
AF811 oro-nasal single-use mask SE, CapStrap headgear, medium	1	1063049	989805626341
AF811 oro-nasal single-use mask SE, CapStrap headgear, large	1	1063048	989805626331
AF811 oro-nasal single-use mask SE, CapStrap headgear, small	5	1063056	989805626411
AF811 oro-nasal single-use mask SE, CapStrap headgear, medium	5	1063055	989805626401
AF811 oro-nasal single-use mask SE, CapStrap headgear, large	5	1063054	989805626391



# Philips Respironics PerformaTrak



PerformaTrak

## Key benefits

- Dual-density foam forehead pads provide patient comfort and mask stability
- With only three sizes, clinicians can quickly fit a wide range of patients
- Swivel clips simplify reapplication by reducing the need to refit after removal

Entrainment elbow (EE)  
Also available with standard elbow (SE)  
For headgear, see page 26

## Philips Respironics PerformaTrak single-use

Description	Quantity	Product number	Part number
PerformaTrak oro-nasal mask EE leak 1, four-point headgear, small	1	1012623	989805610131
PerformaTrak oro-nasal mask EE leak 1, four-point headgear, medium	1	1012624	989805610141
PerformaTrak oro-nasal mask EE leak 1, four-point headgear, large	1	1012572	989805610101
PerformaTrak oro-nasal mask EE leak 1, four-point headgear, small	10	1018592	989805610511
PerformaTrak oro-nasal mask EE leak 1, four-point headgear, medium	10	1018593	989805610521
PerformaTrak oro-nasal mask EE leak 1, four-point headgear, large	10	1018594	989805610531
PerformaTrak oro-nasal mask SE, four-point headgear, small	1	1012636	989805610161
PerformaTrak oro-nasal mask SE, four-point headgear, medium	1	1012637	989805610171
PerformaTrak oro-nasal mask SE, four-point headgear, large	1	1012638	989805610181
PerformaTrak oro-nasal mask SE, four-point headgear, small	10	1018596	989805610551
PerformaTrak oro-nasal mask SE, four-point headgear, medium	10	1018597	989805610561
PerformaTrak oro-nasal mask SE, four-point headgear, large	10	1018598	989805610571
PerformaTrak oro-nasal mask with CapStrap EE leak 1, small	1	1063833	989805626421
PerformaTrak oro-nasal mask with CapStrap EE leak 1, medium	1	1063834	989805626431
PerformaTrak oro-nasal mask with CapStrap EE leak 1, large	1	1063835	989805626441
PerformaTrak oro-nasal mask with CapStrap SE, small	1	1063836	989805626451
PerformaTrak oro-nasal mask with CapStrap SE, medium	1	1063897	989805626461
PerformaTrak oro-nasal mask with CapStrap SE, large	1	1063898	989805626471
PerformaTrak oro-nasal mask with CapStrap EE leak 1, small	10	1019478	989805610631
PerformaTrak oro-nasal mask with CapStrap EE leak 1, medium	10	1019479	989805610641
PerformaTrak oro-nasal mask with CapStrap EE leak 1, large	10	1019480	989805610651
PerformaTrak oro-nasal mask with CapStrap SE, small	10	1019511	989805610751
PerformaTrak oro-nasal mask with CapStrap SE, medium	10	1019525	989805610761
PerformaTrak oro-nasal mask with CapStrap SE, large	10	1019533	989805610771



# Philips Respironics AF421



AF421 with four-point headgear, SE



AF421 EE leak 1



AF421 EE leak 2

## Key benefits

- Supports your infection control efforts with CleanClip
- Oral access feature for oral care
- Leak symbols for ease of ventilator setup
- Available with leak 1-EE, leak 2-EE, and SE elbows for wide ventilator application
- Available in four sizes to fit a wide patient population

## Philips Respironics AF421

Description	Quantity	Product number	Part number
AF421 with EE leak 1 elbow, small*	10 pack	1106162	989805648251
AF421 with EE leak 1 elbow, medium	10 pack	1106163	989805648261
AF421 with EE leak 1 elbow, large	10 pack	1106164	989805648271
AF421 with EE leak 1 elbow, extra large	10 pack	1106165	989805648281
AF421 with EE leak 1 elbow, small*	20 pack	1106166	989805648291
AF421 with EE leak 1 elbow, medium	20 pack	1106167	989805648301
AF421 with EE leak 1 elbow, large	20 pack	1106168	989805648311
AF421 with EE leak 1 elbow, extra large	20 pack	1106169	989805648321
AF421 with SE standard elbow SE, small*	10 pack	1106174	989805648371
AF421 with SE standard elbow SE, medium	10 pack	1106175	989805648381
AF421 with SE standard elbow, large	10 pack	1106176	989805648391
AF421 with SE standard elbow, extra large	10 pack	1106177	989805648401
AF421 with EE leak 2 elbow, small*	10 pack	1106178	989805648411
AF421 with EE leak 2 elbow, medium	10 pack	1106179	989805648421
AF421 with EE leak 2 elbow, large	10 pack	1106180	989805648431
AF421 with EE leak 2 elbow, extra large	10 pack	1106181	989805648441
Accessory, AF421 forehead pad (3/4 in, 19.1 mm)	10 pack	1118318	989805652341
RP, S/M AF421 headgear with clips	5 pack	1109915	989805648851
RP, L/XL AF421 headgear with clips	5 pack	1109916	989805648861
RP, AF421 sizing gauge	25 pack	1109917	989805648871

\* Intended for use by patients 7 years or older and > 40 lb (20 kg)

## Nasal masks

When chronically ill patients enter the hospital in respiratory distress, they typically want an NIV interface as comfortable as what they use at home. They want an unobtrusive system to maintain quality of life during their hospital stay.



### Mask selection criteria

- Oral access
- Long-term NIV







# Philips Respironics Contour Deluxe



### Key benefits

- Dual-density foam forehead pads provide patient comfort and mask stability
- With only three sizes, clinicians can quickly fit a wide range of patients
- Hook-n-loop grab tabs allow quick, easy removal and adjustment of headgear

For headgear, see page 26

### Contour Deluxe

## Philips Respironics Contour Deluxe

Description	Quantity	Product number	Part number
Contour Deluxe nasal mask, petite	5	1016980	989805610311
Contour Deluxe nasal mask, small	5	1016693	989805610251
Contour Deluxe nasal mask, medium/large	5	1016691	989805610231
Contour Deluxe nasal mask, small	10	1016694	989805610261
Contour Deluxe nasal mask, medium/large	10	1016692	989805610241

# Philips Respironics AP111



## Key benefits


- Easy sizing and fitting
- Complete kit for quick setup
- Clear field of vision
- Cradle style nasal cushion

For headgear, see page 26

AP111

## Philips Respironics AP111 single-use

Description	Quantity	Product number	Part number
AP111 interface with cradles* leak 3	1	1062128	989805626481
AP111 interface with cradles* leak 3, circuit, and main flow filter	1	1062130	989805626491
AP111 interface with cradles* leak 3, circuit, and main flow filter	5	1062132	989805626501

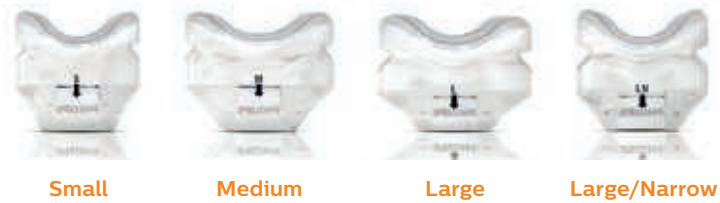

\* Each Philips Respironics AP111 interface includes all four cushion sizes

## Procedure packs and accessories


Get everything you need in one bundle with our procedure packs, designed to simplify the ordering process and help you work more efficiently.

These accessories help you customize your care to suit each individual patient, large or small.



## Procedure packs

Description	Quantity	Product number	Part number
<b>AF531 mask with EE leak 1 nebulizer elbow (no nebulizer), CapStrap headgear, and FEP circuit with main flow filter (#1)</b>			
Small*	5	1084740	989805642171
Medium	5	1084741	989805642181
Large	5	1084742	989805642191
<b>AF531 mask with EE leak 1 entrainment elbow, four-point headgear, and FEP circuit with main flow filter<sup>†</sup></b>			
Small*	5	1072639	989805626511
Medium	5	1072640	989805628811
Large	5	1072641	989805628821
<b>AF811 gel mask with EE leak 1 entrainment elbow, CapStrap headgear, and FEP circuit with main flow filter and exhalation filter<sup>†</sup></b>			
Small	5	1074839	989805633171
Medium	5	1074838	989805633161
Large	5	1074837	989805633151
<b>AF811 gel mask with EE leak 1 entrainment elbow, CapStrap headgear, and FEP circuit with main flow filter<sup>†</sup></b>			
Small	5	1074061	989805633201
Medium	5	1074060	989805633191
Large	5	1074059	989805633181
<b>PerformaTrak mask with EE leak 1 entrainment elbow, four-point headgear, and FEP circuit with main flow filter<sup>†</sup></b>			
Small	5	1074055	989805633111
Medium	5	1074054	989805633101
Large	5	1074053	989805633091
<b>PerformaTrak mask with EE leak 1 entrainment elbow, CapStrap headgear, and FEP circuit with main flow filter<sup>†</sup></b>			
Small	5	1074058	989805633141
Medium	5	1074057	989805633131
Large	5	1074056	989805633121

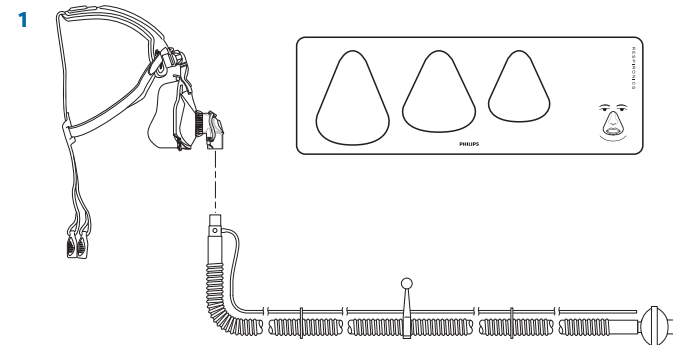

\* Intended for use by patients 7 years or older and > 40 lb (20 kg)

<sup>†</sup> Not illustrated

Note: FEP – Filter Exhalation Port



# Aerogen Pro-X controllers and systems



Aerogen Pro-X controller


## NIVO Pro-X controller kits\*

Description	Quantity	Product number	Part number
NIVO Pro-X controller kit – US	1	1076301	989805634291
NIVO Pro-X controller kit – Canada	1	1082697	989805634511

## NIVO nebulizers and elbows

Description	Quantity	Product number	Part number
NIVO nebulizer (#7)	5	1076302	989805634301
NIVO nebulizer, EE, leak 1 entrainment elbow (#6)	5	1084320	989805635031
NIVO nebulizer, SE standard elbow (#6)	5	1084323	989805635041
AC/DC adapter – US (#5)	1	1022696	989805612461
ProX Pole Mount (#3)	1	1022693	989805614101
Mount adapter for humidifier mounting slot (#8)	1	1022692	989805614261
Control module cable (#5)	1	1022706	989805613361

# NIVO controller and system components

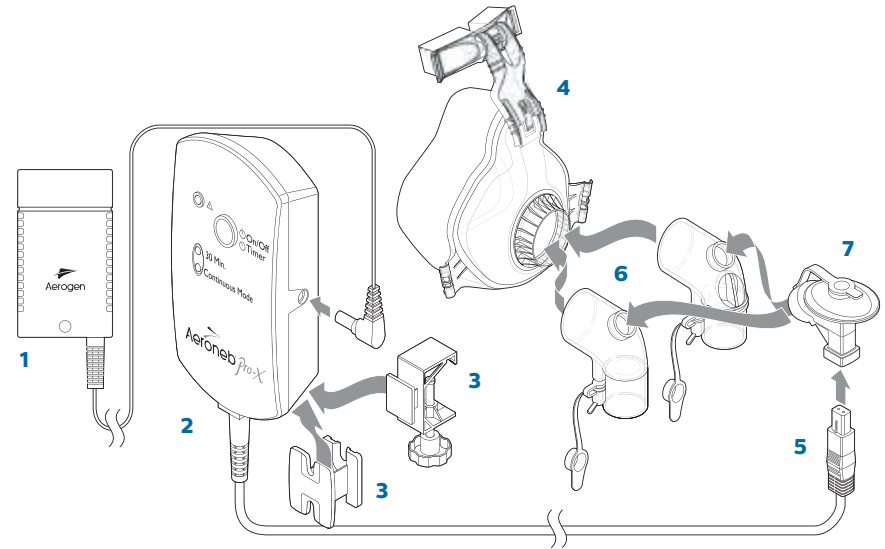

- 1 AC/DC adapter Power source or charge controller
- 2 Pro-X control module Can use AC/DC adapter or internal battery (up to 45 minutes)
- 3 Equipment mount Use for pole or humidifier slot mounting configuration
- 4 AF531 mask\*\* Mask to be used
- 5 Control module cable Connect controller to NIVO nebulizer
- 6 Nebulizer elbows (NE) Entrainment elbow (EE) or standard elbow (SE) with nebulizer port
- 7 NIVO nebulizer Aerosol generator
- 8 Mount adapter Mount adapter for humidifier mounting slot

\* Each kit contains: Power source, Aerogen Pro-X, equipment mount, and control module cable.  
 \*\* NE elbow can be added to an existing AF531 or mask can be purchased with NE elbow installed.

# Mask accessories

## Philips Respironics AF531 accessories and replacement parts

Description	Quantity	Product number	Part number
EE leak 1 entrainment elbow*	1	1072645	989805628841
EE leak 1 entrainment elbow*	5	1072646	989805628831
EE leak 1 nebulizing elbow (no nebulizer)*	1	1084324	989805635051
EE leak 1 nebulizing elbow (no nebulizer)*	5	1084325	989805635061
SE nebulizing elbow (no nebulizer)*	1	1084326	989805635071
SE nebulizing elbow (no nebulizer)*	5	1084327	989805635081
NIVO nebulizer	5	1076302	989805634301
CapStrap headgear with clips	1	1072654	989805628871
Four-point headgear with clips, small	1	1072655	989805628891
Four-point headgear with clips, large	1	1072656	989805628881
Four-point headgear with clips, extra large	5	1083188	989805634751
AF531 sizing gauge	25	1078287	989805628901
Swivel	1	7041	453561517051
RP-PMax and AF531 headgear clips	2	1060913	989805649921
Nasogastric sealing pad	10	452040	453561517091
CleanClip bracket assembly	1	1076430	989805627181
CleanClip bracket thumb screw	1	1076197	989805629911
Bronchoscopy elbow*	1	1072651	989805635091
Bronchoscopy elbow*	5	1072652	989805635101

## Philips Respironics PerforMax accessories and replacement parts

Description	Quantity	Product number	Part number
EE leak 2 entrainment elbow*	1	1045313	989805611111
EE leak 2 entrainment elbow*	5	1072647	989805628851
Bronchoscopy elbow*	1	1072651	989805635091
Bronchoscopy elbow*	5	1072652	989805635101
SE standard elbow*	1	1043210	989805611081
SE standard elbow*	5	1072648	989805628861

\* Elbow – Conical type: Push and twist to install. Pull and twist to remove. Refer to instructions with compatible mask or elbow for proper usage. Conical type elbows used on AF531, PerforMAX and PerforMAX pediatric mask.


## Philips Respironics AF541 accessories and replacement parts

Description	Quantity	Product number	Part number
AF541 over-the-nose cushion, small	10	1120960	989805654131
AF541 over-the-nose cushion, medium	10	1120961	989805654141
AF541 over-the-nose cushion, large	10	1120962	989805654151
AF541 over-the-nose cushion, extra large	10	1120963	989805654161
EE leak 1 elbow with NIVO port**	5	1120967	989805654171
EE leak 2 elbow with NIVO port**	5	1120968	989805654181
SE standard elbow with NIVO port**	5	1120969	989805654191
AF541 EE leak 1, without cushion, four-point headgear	5	1120975	989805654201
AF541 EE leak 1, without cushion, CapStrap headgear	5	1120976	989805654211
AF541 four-point headgear with forehead pad, small/medium	5	1120970	989805654221
AF541 four-point headgear with forehead pad, large/extra large	5	1120971	989805654231
AF541 CapStrap headgear, small/medium	5	1126500	989805654251
AF541 CapStrap headgear, large/extra large	5	1120972	989805654241
AF541 sizing gauge	25	1120977	989805654261
AF541 size A under-the-nose cushion	10	1120980	989805654271
AF541 size B under-the-nose cushion	10	1120981	989805654281
AF541 size C under-the-nose cushion	10	1120982	989805654291
Bronchoscopy elbow	5	1120984	989805654301
AF541 discharge kit, CapStrap headgear, small/medium	5	1120986	989805654311
AF541 discharge kit, four-point headgear, small/medium	5	1120987	989805654321
AF541 discharge kit, CapStrap headgear, large/extra large	5	1120988	989805654331
AF541 discharge kit, four-point headgear, large/extra large	5	1120989	989805654341

\*\* Elbow – Snap/Squeeze type: Push for audible click to install and squeeze tabs to remove. Refer to instructions with compatible mask or elbow for proper usage. Snap/Squeeze type used on AF541 mask.

## Circuit and mask components and accessories


Image	Description	Quantity	Product number	Part number
<b>1</b>	Plateau exhalation valve (reusable)	1	302312	989805617941
<b>2</b>	Plateau exhalation valve diaphragm	5	302310	989805609221
<b>3</b>	Whisper Swivel II	1	332113	989805617951
<b>4</b>	Disposable exhalation port with filter exhalation port with swivel (no filter)	10	1065775	453561517211
<b>4a</b>	Replacement port caps (silicone)	5	1061825	453561517201
<b>4b</b>	Disposable exhalation port with filter exhalation port and proximal pressure line (7 ft, 2.1 m)	10	1116891	989805652291
<b>4b</b>	Disposable exhalation port with filter exhalation port and proximal pressure line (7 ft, 2.1 m)	1	1116892	989805652301
<b>5</b>	Disposable exhalation port	10	312149	989805609391
<b>6</b>	Oxygen enrichment attachment	1	312710	989805609421
<b>6</b>	Oxygen enrichment attachment	10	312010	989805609241
<b>7</b>	Bacteria/virus filter (single use)	1	1014047	989805618161
<b>7</b>	Bacteria/virus filter (single use)	10	342077	989805609521
<b>8</b>	Proximal airway filter	1	1002362	453561517101
<b>9</b>	Tube management clip	2	1039967	453561517351

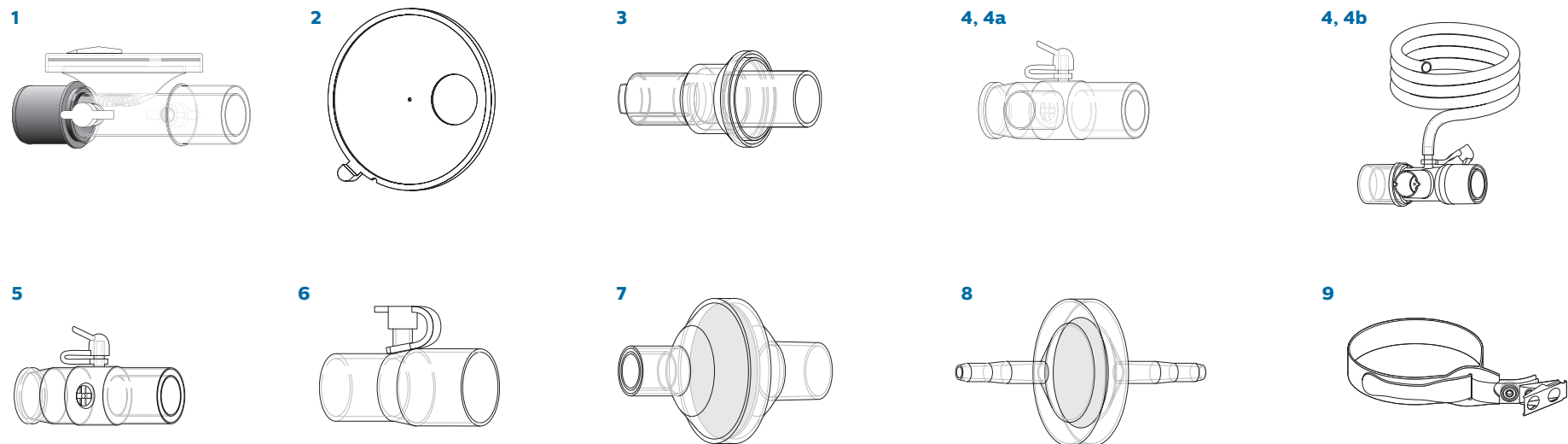


## Circuit and mask components and accessories

Image	Description	Quantity	Product number	Part number
<b>10</b>	Swivel replacement kit (22 mm male-male)	1	7041	453561517051
<b>11</b>	CleanClip bracket assembly	1	1076430	989805627181
<b>11</b>	CleanClip bracket thumb screw	1	1076197	989805629911
<b>12</b>	Nasogastric tube sealing pad	10	452040	453561517091
<b>13</b>	PerforMax sizing gauge, small, large, and extra large	1	1095097	989805648901
<b>14</b>	Replacement ball and socket	2	1009723	989805652511


**10**

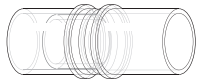


**11**

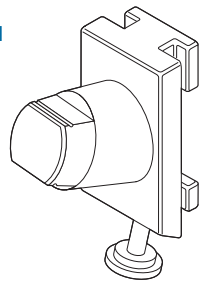


**12**

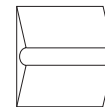

**13**

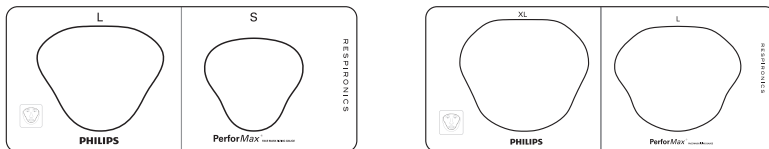


**14**

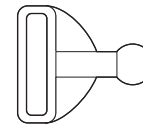

\* Image not shown

## Headgear

Image	Description	Size	Compatibility	Quantity	Product number	Part number
1	CapStrap headgear with ball and socket clips	Adult (one size)	AF811, PerformaTrak	5	1019543	989805610841
1	CapStrap headgear with clips	Adult (one size)	AF531	1	1072654	989805628871
2	Four-point headgear with ball and socket clips	Adult (one size)	AF811, PerformaTrak	1	1015788	989805610221
3	Four-point headgear, no clips	Petite	AF811, PerformaTrak Contour Deluxe	5	1018472	989805610351
3	Four-point headgear, no clips	Adult (one size)	AF811, PerformaTrak Contour Deluxe	5	1007365	989805609961
4	Four-point headgear	Adult (one size)	Total	1	1003052	453561517111
4	Four-point headgear	Adult (one size)	Total	9	1010609	989805610031
5	Four-point headgear	Adult (one size)	Vinyl nasal	10	312106	989805609291
6	Four-point headgear with crown strap, clips	Small	PerforMax	5	1051857	989805611231
6	Four-point headgear with crown strap, clips	Large	PerforMax	5	1051816	989805611221
6	Four-point headgear with crown strap, clips	Extra large	PerforMax	5	1080518	989805634851
6	Four-point headgear with crown strap, clips	Small	PerforMax	9	1045325	989805611131
6	Four-point headgear with crown strap, clips	Large	PerforMax	9	1045314	989805611121
6	Four-point headgear with crown strap, clips	Extra large	PerforMax	9	1080519	989805634861
6	Four-point headgear with crown strap, clips	Small	AF531	1	1072655	989805628891
6	Four-point headgear with crown strap, clips	Large	AF531	1	1072656	989805628881
6	Four-point headgear with crown strap, clips	Extra large	AF531	5	1083188	989805634751
7	Two-point headgear with chin strap	Adult (one size)	AP111	5	1062134	453561516551
8	AF421 forehead pad kit with two pads	10 mm	AF421	10	1119366	989805652521
9	AF541 four-point headgear with forehead pad	Small/Medium	AF541	5	1120970	989805654221
9	AF541 four-point headgear with forehead pad	Large/Extra large	AF541	5	1120971	989805654231
10	AF541 CapStrap headgear	Small/Medium	AF541	5	1126500	989805654251
10	AF541 CapStrap headgear	Large/Extra large	AF541	5	1120972	989805654241

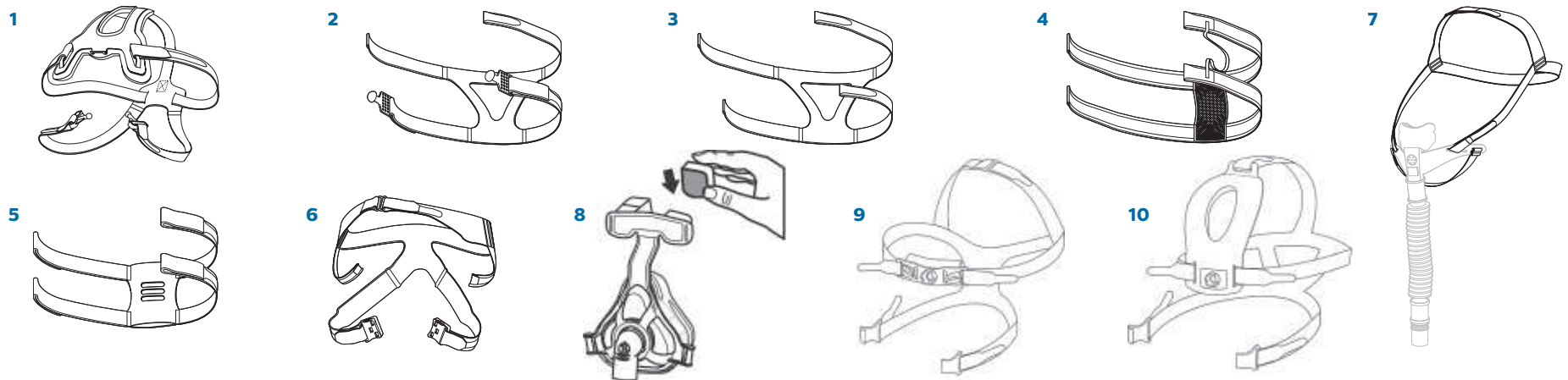










# Circuits and connections

Philips offers a variety of single-use circuits, including heated-wire and non-heated-wire options, to accompany our noninvasive ventilation systems.

## Circuits

Image	Description	Product number	Part number	Quantity	V60	BiPAP Vision	Trilogy 202	BiPAP Focus	BiPAP S/T-D
1	Invasive circuit with exhalation port, water trap and temperature probe ports, proximal airway filter and humidifier coupling tube, hanger, and hose clips	652002	989805609681	10	•	•	•		
1	Invasive circuit with exhalation port, water trap and temperature probe ports, proximal airway filter and humidifier coupling tube, hanger, and hose clips	652001	989805609671	20	•	•	•		
2	Single limb (25 mm) heated-wire circuit with exhalation port	1020523	989805610851	10	•	•	•		
3	Noninvasive circuit with main flow filter, disposable swivel exhalation port with filter port and filter, proximal pressure line, hanger, and hose clips	1065830	989805621311	10	•	•	•	•	
4	Noninvasive circuit with main flow filter, disposable swivel exhalation port with filter port, proximal pressure line, hanger, and hose clips (exhalation filter not included)	1065832	989805621321	10	•	•	•	•	
5	Noninvasive circuit with disposable swivel exhalation port with filter port, proximal pressure line, hanger, and hose clips (main flow and exhalation filters not included)	1069210	989805634871	10	•	•	•	•	
6	Noninvasive circuit with disposable exhalation port, proximal pressure line, hanger, and hose clips	582073	989805609611	10	•	•	•	•	
7	BiPAP S/T-D circuit with exhalation port, proximal pressure line, hanger, and hose clips	312107	989805609301	10					•
8	Tubing 6 ft (22 mm) (no dep, proximal pressure line, or hanger)	532316	989805647181	1					

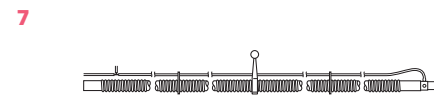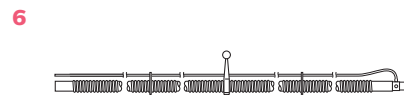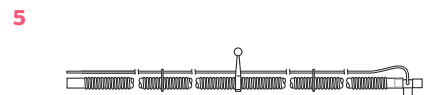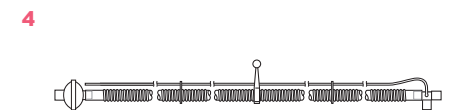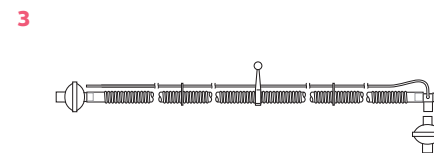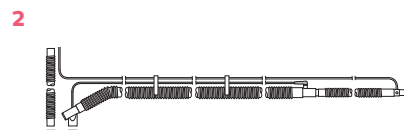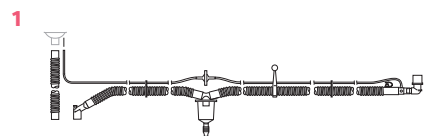

## Trilogy accessories

The innovative technology of Trilogy sets it apart from the rest. The support, service, and upkeep behind it keep it there.

### Easy to use and maintain

- Trilogy circuits can be used for a variety of pediatric and adult patient applications
- Passive circuits for NIV applications
- Active exhalation valve for IV application
- Circuits are available in packs of 10 and 20




## Circuit adapters


Image	Description	Compatibility	Product number	Part number
1	Active exhalation valve with PAP assembly		1065659 – 1/pk	989805628221
2	Proximal flow sensor (Trilogy202)		1050408 – 1/pk 1050534 – 10/pk	989805621211 989805645511
3	Disposable swivel passive exhalation port		1084428 – box of 10 1084429 – box of 20	989805643161 989805646961
4	Flexible trach adapter with 15 mm cuff	Connects to the flow sensor or the active exhalation valve	1073863 – box of 10 1073903 – box of 20	989805633921 989805633941
5	Flexible trach adapter with 22 mm cuff	Connects to Whisper Swivel II and swivel and fixed disposable exhalation ports	1073902 – box of 10 1073904 – box of 20	989805633931 989805633951

The ability to use either an active valve or passive exhalation port gives you the flexibility to choose the best available circuit and interface for your patient. Trilogy's unique passive exhalation port reduces extra tubing connections.


1

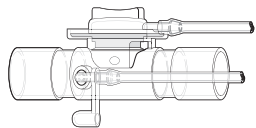


2

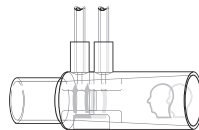


3

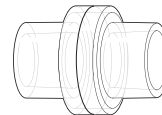

4

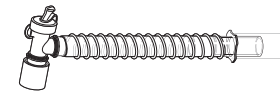


5

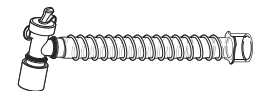


## Miscellaneous

Image	Product	Product number	Part number
6	Oxygen inlet adapter – 2/pk	1040390	989805647691
7	Tube port adapter, active PAP	1061398	989805621291

6

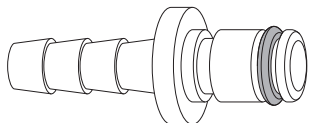


7

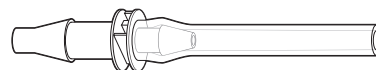

## Trilogy support arm system

Image	Product	Product number	Part number
8	Tubing hanger (22 mm) – 1/pk	311107	989805648891
*	Support arm	332497	989805617961
*	Support arm mount assembly	1052863	989805605731

8

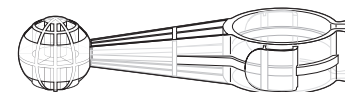

\* Image not shown


## Trilogy disposable circuits

Image	Description	Components	Quantity	Product number	Part number
<b>1</b>	Adult passive circuit with fixed exhalation port	Passive exhalation port. Patient tube: 6 ft length, 22 mm diameter with 22 mm cuff.	Box of 10	1069467	989805621551
			Box of 20	1069468	989805621561
<b>2</b>	Adult passive circuit with swivel design exhalation port	Passive exhalation port. Patient tube: 6 ft length, 22 mm diameter with 22 mm cuff.	Box of 10	1073228	989805633861
			Box of 20	1073236	989805633911
<b>3*</b>	Adult passive circuit with water trap	Standard 90° degree elbow adapter. Passive fixed exhalation port. Temperature probe port. Patient tube, 2 ft length, 22 mm diameter with 22 mm cuff. Water trap. Humidifier connection tube, 2 ft length, 22 mm diameter with 22 mm cuff.	Box of 10	1069443	989805621531
			Box of 20	1069444	989805621541
<b>4*</b>	Adult passive circuit, heated wire	Passive swivel exhalation port. Temperature probe port. Heated wire patient tube, 5 ft length, 22 mm diameter with 22 mm cuff. Heated wire adapter. Humidifier connection tube, 2 ft length, 22 mm in diameter with 22 mm cuff.	Box of 10	1073227	989805646441
			Box of 20	1073235	989805646481
<b>5</b>	Adult active circuit, no water trap	Standard 90° elbow trach adapter. Active exhalation valve. Exhalation tubing. Proximal pressure tubing. Patient tube, 6 ft length, 22 mm diameter with 22 mm cuff. Tube management clips.	Box of 10	1073223	989805633841
			Box of 20	1073232	989805633881
<b>6*</b>	Adult active circuit, heated wire	Standard 90° elbow trach adapter. Active exhalation valve. Exhalation tubing. Proximal pressure tubing. Heated wire patient tube, 5 ft length, 22 mm diameter with 22 mm cuff, with heated wire connector. Heated wire adapter. Temperature probe port. Humidifier connection tube, 2 ft length, 15 mm diameter with 22 mm cuff. Tube management clips.	Box of 10	1073222	989805646421
			Box of 20	1073231	989805646461


1




2

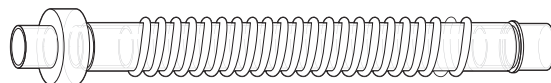


3

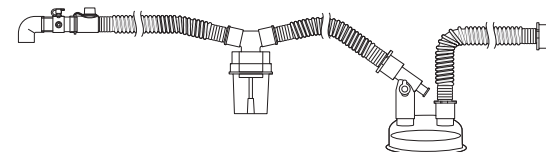


4

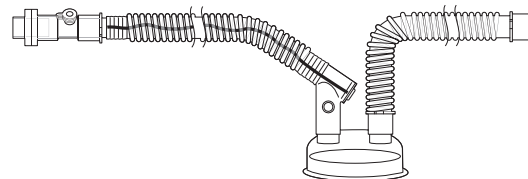

5

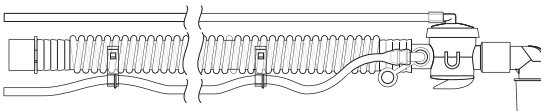

6

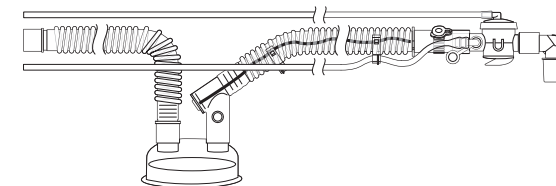


\* Chamber not included


## Trilogy disposable circuits

Image	Description	Components	Quantity	Product number	Part number
<b>7</b>	Pediatric passive circuit with water trap	Passive fixed exhalation port. Patient tube, 6 ft length, 15 mm diameter with 22 mm cuff and water trap.	Box of 10 Box of 20	1069445 1069446	453561517231 453561517241
<b>8*</b>	Pediatric passive circuit with water trap	Passive swivel exhalation port. Patient tube, 2.5 ft length, 15 mm diameter with 22 mm cuff. Temperature probe port. Water trap. Humidifier connection tube, 2 ft length, 15 mm diameter with 22 mm cuff.	Box of 10 Box of 20	1073224 1073233	989805633851 989805633901
<b>9*</b>	Pediatric passive circuit, heated wire	Passive swivel exhalation port. Heated wire patient tube, 5 ft length, 15 mm diameter with 22 mm cuff, with heated wire connector. Heated wire adapter. Temperature probe port. Humidifier connection tube, 2 ft length, 15 mm diameter with 22 mm cuff.	Box of 10 Box of 20	1073225 1073234	989805646431 989805646471
<b>10*</b>	Pediatric active circuit with water trap	Standard 90° elbow trach adapter. Active exhalation valve. Exhalation tubing. Proximal pressure tubing. Patient tube, 2.5 ft length, 15 mm diameter with 22 mm cuff. Temperature probe port. Water trap. Humidifier connection tube, 2 ft length, 15 mm diameter with 22 mm cuff. Tube management clips (one shown).	Box of 10 Box of 20	1073220 1073229	989805633831 989805633871
<b>11*</b>	Pediatric active circuit, heated wire	Standard 90° elbow trach adapter. Active exhalation valve. Exhalation tubing. Proximal pressure tubing. Heated wire patient tube, 5 ft length, 15 mm diameter with 22 mm cuff, with heated wire connector. Heated wire adapter. Temperature probe port. Tube management clips. Humidifier connection tube, 2 ft length, 15 mm diameter with 22 mm cuff.	Box of 10 Box of 20	1073221 1073230	989805646411 989805646451
<b>12</b>	MPV (mouthpiece ventilation) circuit, single use	Bacteria filter, 22 mm x 15 mm reducer cuff, 6 ft length, 15 mm tubing, coupler, miniature and angled mouthpiece.	Box of 10	1102866	989805646981


7

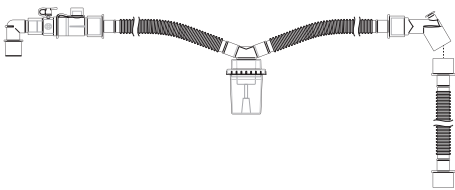


8

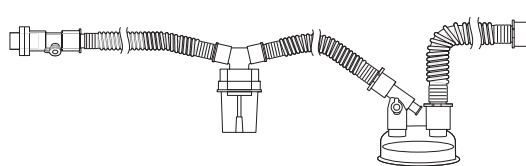


9

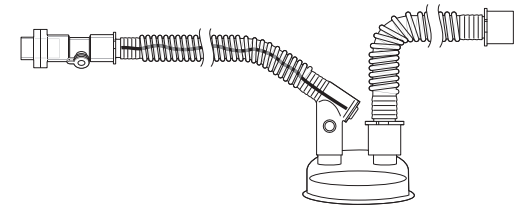


10

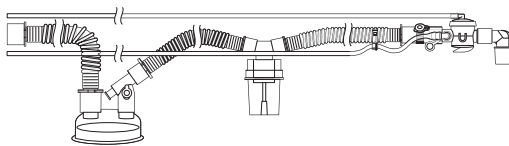

11

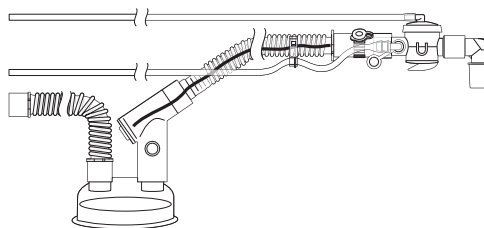

12

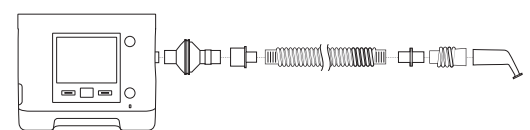

\* Chamber not included



# Notes

©2016 Koninklijke Philips N.V. All rights are reserved.  
Philips reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication. Trademarks are the property of Koninklijke Philips N.V. or their respective owners.



[www.philips.com/NIVmasks](http://www.philips.com/NIVmasks)

Printed in the Netherlands.  
4522 991 16531 \* AUG 2016